

CASE STUDY

Patient CB K Readings 7.90 / 7.85 BC 7.85 (0.05 steeper than the Flat K)




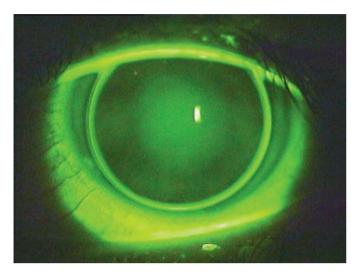


TRICURVE 7.60/9.50 Very steep, looks only a little steep but fluorescein not getting under as it is sealed off.

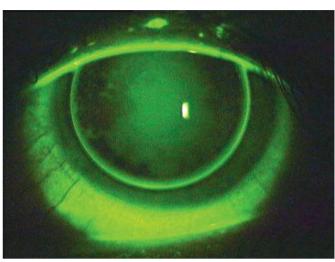


TRICURVE 7.70/9.50 Still steep.

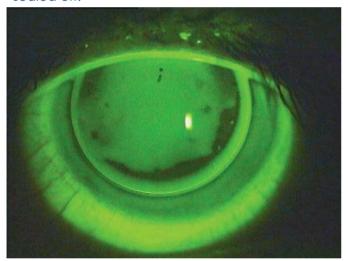

TRICURVE 7.80/9.50 About the correct base curve but photo shows fluorescein on front surface of the lens.

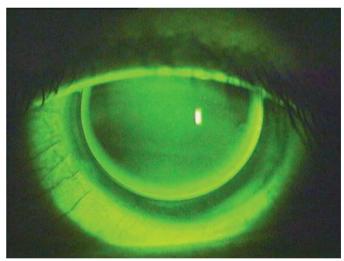

TRICURVE 7.90/9.50 Good fit.

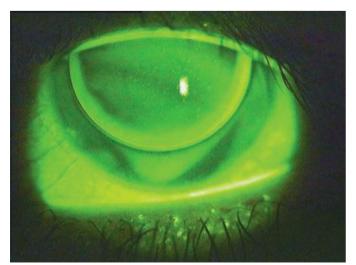

TRICURVE 8.00/9.50 Flat fit and lens is starting to ride higher.